ISO 9001:2000 JAS-ANZ VNZ

ISO 13485

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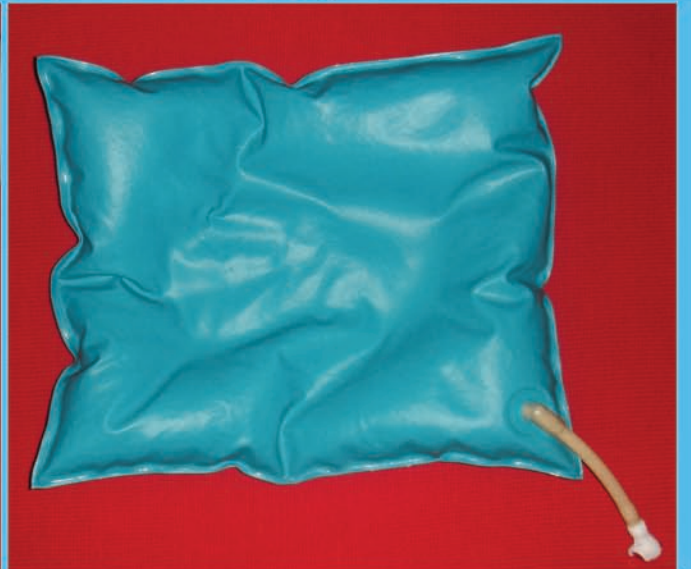
#### ARGIFORM Vacuumed Surgical Positioning Mattress & Cushion

Its fixation feature enables optimum admission to surgeon for surgical operation area by vacuuming its "Safe Application" System.

- Patient position is provided reliably and in a short period.
- Excessive grading of operation table does not impair ArgiForm's area pressure distribution and the patient position.
- When it is desired to change patient position, the application can be made easily and in a short time.
- The patient can be transferred between clinics without changing the position of patient.
- Safe Fixation
- Excessive Grading Tolerance
- Fast and Easy Application

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## ArgiForm Vacumed Surgical Position Mattress and Cushion:

- Argiform is a product created by technological and scientific studies of physiotherapists, dermatologists, surgeons and anatomy experts as a result of laboratorial and experimental studies for a long period as well as research and development performed.
- Argiform provides stabilization, temperature maintenance and micro-circulation required by skin and tissues by means of millions of special polystyrene particles inside.
- Argiform minimizes contact pressure of patient body and provides this pressure to be distributed equally to the all spots of body.
- Argiform accords with the body of patient ergonomically and preserves body temperature.
- Argiform removes complication risk by meeting postoperative and long-period position comfort utilizations in all surgical positions and it prevents deformatio.
- Argiform makes doctor, patient and operation team to gain safety and time.  
It provides Maximum Fixation comfort.  
It prevents decubitus skin ulceration.  
It preserves body temperature.  
It prevents complications.  
It provides patient and user comfort.

## How ArgiForm Works?

ArgiForm surgical position mattresses are formed of 3 dimensioned special PVC and special micro granules and presented for the service of medicine and health sector by being designed as preventing to slip. By the quality of same pressure and of taking the shape of each part of body individually and by having special Polystyren granules which are stabilized together when vacuumed, it performs its positioning and temperature preservation duties successfully. ArgiForm does not contain latex and it is ideal for having CT and MR since it transmits X-RAY.

## Using ArgiForm:

- ArgiForm is spread over operation table.
- When desired, sterile drape or sterile green cover can be spread over.
- Optionally, it is cleaned with disinfectant.
- Desired position is given to patient on ArgiForm.
- ArgiForm is shaped as to the position of patient.
- ArgiForm hose is connected to central vacuum system or vacuum pump. The valve is kept open.
- The stability of patient position is provided now.
- The valve is brought to OFF position.
- When it is desired to change the position of patient or when the operation is finished, vacuum switch is opened again to give it free form. It does not include latex.  
It is suitable for having CT and MR.  
It has extensive product options.

## ArgiForm Vacumed Mattress

ArgiForm meets all surgical procedures required for positioning patient distinctively. There is always an ArgiForm variety for all kinds of surgical position. When desired, special size manufacturing can be done. You can see samples and their sizes on the side page.

- Side Position
- Half-Side Position
- Back Position
- Sitting-Half Sitting Position
- Bend Position
- Lithotomy Position
- Laparoscopic Position

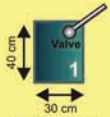
- Argiform option appropriate for all clinics
- Product variety appropriate for personal requirements

## ArgiForm Vacumed Operation Cushion


It has the all advantages of Surgical Position Mattress; it is just designed for head area positions. It is ideal for fixation of Head or Neck area positions.

## ArgiForm Pump


In hospitals not having vacuum systems, vacuum can be made to the admission area by means of an appropriate pump.

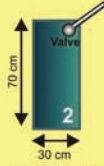
Argi Form  
Surgical Cushion  
Cerrahi Yastık  
30 cm x 40 cm  
ARCC015034

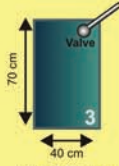
Argi Form  
Surgical Cushion  
Cerrahi Yastık  
30 cm x 70 cm  
ARCC015037

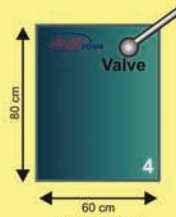
Argi Form  
Surgical Cushion  
Cerrahi Yastık  
40 cm x 70 cm  
ARCC015047

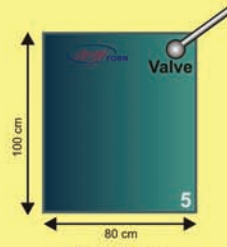
Argi Form  
Surgical Mattress  
Cerrahi Yatak  
60 cm x 80 cm  
ARCC015068

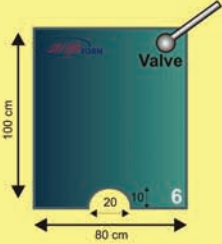
Argi Form  
Short Surgical Mattress  
Gynecology Pediatrics  
Kısa Cerrahi Yatak  
Jinekoloji Pediatri  
80 cm x 100 cm  
ARMJ015081

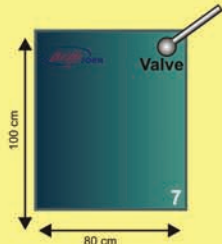
Argi Form  
Short Cavernous  
Surgical Mattress  
Gynecology Urology  
Kısa Oyuklu Cerrahi Yatak  
Jinekoloji Üroloji  
80 cm x 100 cm  
ARMU015081

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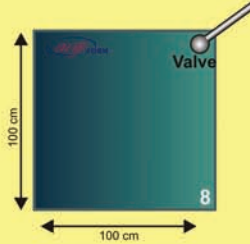
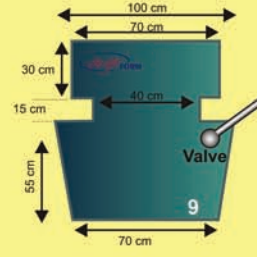
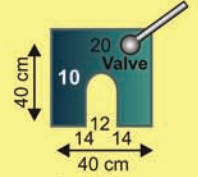
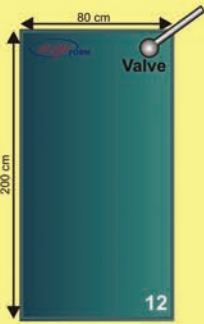
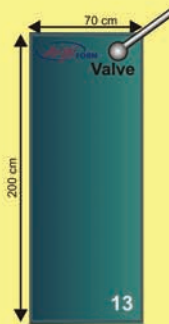
Argi Form  
Universal Surgical Mattress  
Universal  
Cerrahi Yatak  
80 cm x 200 cm  
ARMU015082

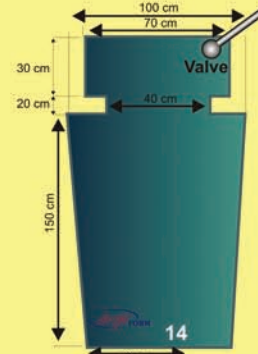
Argi Form  
Universal Surgical Mattress  
Universal  
Cerrahi Yatak  
70 cm x 200 cm  
ARMU015072

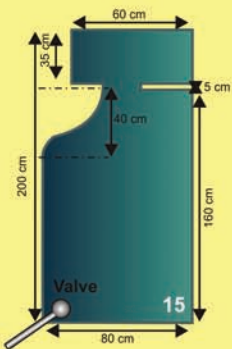
Argi Form  
Custom Formed Surgical Mattress  
Side Position  
Özel Şekilli Cerrahi Yatak  
Yan Pozisyon  
100 cm / 70 cm x 200 cm  
ARMY015012

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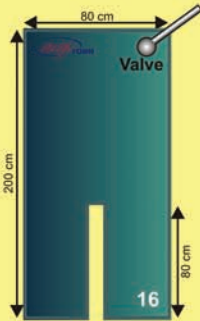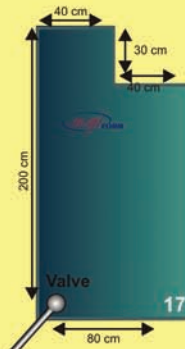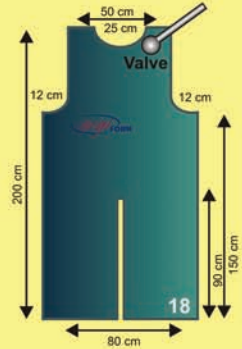
Argi Form  
Custom Formed  
Surgical Mattress  
With Leg and Neck Cavity  
Özel Şekilli Cerrahi Yatak  
Bacak ve Boyun oluklu  
80 cm x 200 cm  
ARMB015082

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## What is Decubitus?

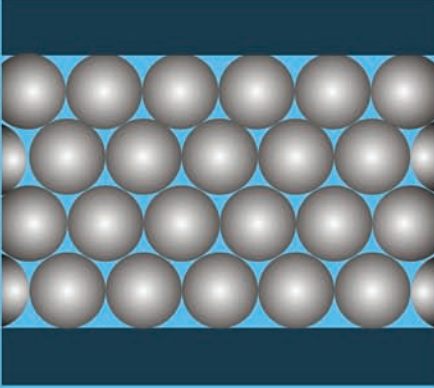
Decubitus is a damage form caused by a longtime pressure onto skin and tissues under skin. It is proved in scientific studies that decubitus risk appears after 10-20 minutes under increasing pressure. It means pain, insulation and decrease of life quality for patient.

## Positioning on Operation Table and Decubitus


Especially during surgical processes, Acute Decubitus skin ulceration, paresis and bed pain risks appear. This risk is high on all longtime surgical processes requiring position.

## Safety and Comfort on Operation Table



During operation, safety and comfort of patient and doctor are provided by means of Position given to patient by Argi Form. Argi Form reliably helps resulting surgical interventions and decreasing risks.

Vakumsuz  
not vacuumed

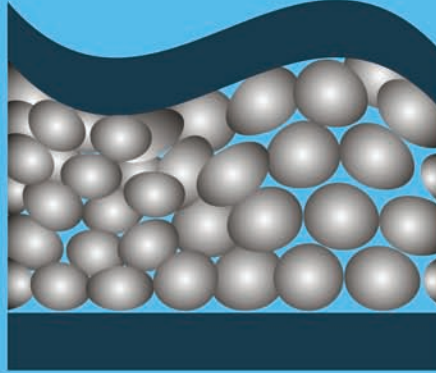Vakumlu  
vacuumed
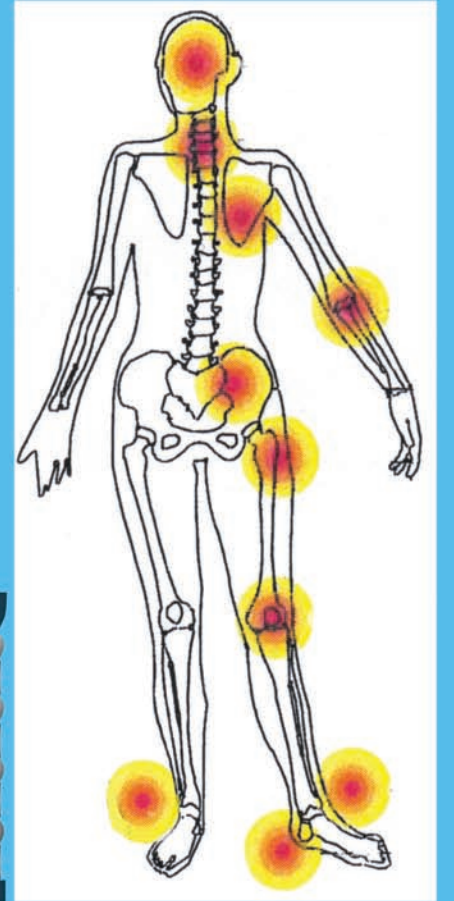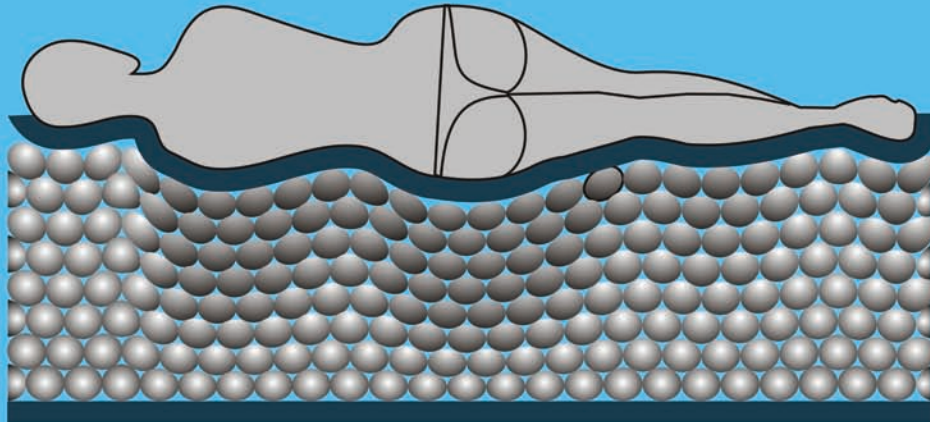
## Üretici Manufacturer

**argi** GROUP

Sağlık-1Sk. Çakırgil Apt. 7/8 06420-07 Yenışehir/ANKARA - TURKEY  
Tel : +90 312 430 65 18 -19 Fax : +90 312 430 41 04  
e-mail : info@argigrup.com • Web : www.argigrup.com